



PAL Health Technologies

...it's about TIME!



PAL Health Technologies

...it's about TIME!

Platinum[®] Collection

PREMIUM ANKLE BRACES

***Patient Warranty
& Fitting Instructions***

CORPORATE OFFICE 1805 Riverway Drive | Pekin, IL 61554
PHONE 800.447.0151 | FAX 877.957.5647 | www.palhealth.com

PAL175 (8/06)



THANK YOU for choosing the Platinum® Ankle Brace. This Ankle Support System is designed to provide superior ankle stabilization when used in conjunction with a lace-up shoe.

PAL Health Technologies warrants to the purchaser that the Platinum® Ankle Brace will be free from defects in material and workmanship for ninety (90) days from the date of delivery to the original purchaser under the normal use in accordance with directions. Please refer all returns to your prescribing foot care professional.

FITTING THE PLATINUM® ANKLE BRACE

While it fits into many shoe styles, the Platinum® Ankle Brace is not designed to provide support in soft-sided shoes, hi-top boots, or sandals.

The following shoes are recommended: Rockport Pro Walker, Ecco, Munroe, and most athletic shoes. Just follow these simple instructions for fitting the brace into your shoe. **DO NOT** attempt to use this brace without a shoe.

1. To apply the Platinum® Ankle Brace, place it in the shoe and loosen all four velcro straps.
2. Open the brace, tilt the limb supports back, and slide your foot in.
3. After your foot is in, lift your foot slightly while sliding the Platinum® Ankle Brace toward the back of the shoe to assure proper placement.
4. Adjust the straps for maximum comfort. If straps are too long at the rivet behind the buckle, pull the hook velcro away from the loop and trim the padded velcro at the sonic welded intervals (1-1/2" intervals).

5. To remove, loosen all straps, open the brace, tilt the limb supports back and remove your foot.
6. Braces equipped with viscoelastic polymer gel lower pads are also designed to be used for either hot or cold therapy. Remove the pads from the brace and place in the freezer for 15–30 minutes for cold therapy. For hot therapy, remove the pads from the brace and place pads in the microwave for approximately 5–10 seconds only.

CAUTION: DO NOT OVERHEAT.

CLEANING

The plastic foot plate and limb supports may be cleaned using a mild, soapy water solution and a soft cloth. Avoid usage of strong caustic cleaning solutions as they may weaken the plastic.

***For questions or
customer service, call:
1.800.447.0151***

